Laparoscopic TUBING SOLUTIONS

BLAP100

- Easy to use
- Sterile
- Latex-Free
- Surgical Drape Clasp
- Luer Connector for Trocar/Cannula

Easily connects to trocar and smoke evacuator to allow for continual removal of smoke plume.

Buffalo Filter's BLAP tubing options are designed for laparoscopic procedures. These tubing sets allow surgical staff to quickly and efficiently clear smoke from the peritoneal cavity during laparoscopic procedures.

Directions For Use

The following protocol should be used for set-up and operation for the BLAP100V.

- Connect patient end Luer Connector on the tubing to the appropriate surgical device (stopcock on Trocar or Canula).
- Connect large open end of tubing to Buffalo Filter Smoke Evacuator filter inlet port. If fluid will be evacuated, please use fluid suction container (P/N SCC1001). See equipment Operator's manual for additional details
- Position the drape clip. Secure onto tubing.
- When smoke plume needs to be evacuated, turn on the Buffalo Filter Smoke Evacuation System and then turn the stopcock on the trocar to evacuate the smoke from the surgical site.
- This Laparoscopic Smoke Evacuator Tubing Kit can be used at 10-50% speed settings with the following:
 - Porta PlumeSafe® 603
 - PlumeSafe® Whisper™
 - PlumeSafe® Whisper™ TURBO
 - PlumeSafe® Auto-Sense

CAUTION: Use the lowest possible setting for removal of smoke plume from the surgical site. Do not exceed 50% motor speed during surgical procedure.

CAUTION: Extended smoke removal by the stopcock on the trocar may cause reduction of the pneumoperitonium.

If you have any additional questions about using this product please contact your Sales Representative or Buffalo Filter directly.

Product Ordering Information

PART # BLAP100 12/ Case

Laparoscopic/Endoscopic Smoke Evacuation Tubing Kit

Kit contains: Connector to 7/8" (22mm) filter inlet, 1/4" (6mm) x 10' (305cm) tubing w/ Luer Connector adapter for Trocar. On/Off controlled by trocar stopcock or RSA1002 (available separately). Surgical Drape Clasp Attached. Sterile Kit.

Buffalo Filter®

595 Commerce Drive
Buffalo, NY 14228
toll-free: 800.343.2324
local: 716.835.7000
fax: 716.835.3414

e-mail: info@buffalofilter.com
www.buffalofilter.com

INSTRUCTIONS FOR USE

PLEASE CHECK PRODUCT LABEL AND FOLLOW APPROPRIATE INSTRUCTIONS BELOW

Sterile Unless Unit Package Damaged or Opened – Single Use – Disposable

Laparoscopic Smoke Evacuator Tubing Kit Part Number BLAP100

1. Connect patient end Luer Connector on the tubing to the appropriate surgical device (stopcock on trocar or canula).
2. Connect large open end of tubing to Buffalo Filter® Smoke Evacuator filter inlet port. If fluid will be evacuated, please use fluid suction container (P/N SCC1001).
3. Position the drape clip.
4. When smoke plume needs to be evacuated, depress the suction on/off button on the Buffalo Filter® Smoke Evacuation System and then turn the stopcock on the trocar to evacuate the smoke from the surgical site.
5. This Laparoscopic Smoke Evacuator Tubing Kit should be used at approximately 10-50% suction control level with any Buffalo Filter Smoke Evacuation System.

CAUTION: Use the lowest possible setting for removal of smoke plume from the surgical site. Do not exceed 50% suction control level during surgical procedure.

CAUTION: Extended smoke removal by the stopcock on the trocar may cause reduction of the pneumoperitoneum.

Laparoscopic Smoke Evacuator Tubing Kit With Control Valve Part Number BLAP100V

1. Connect patient end Luer Connector on the tubing to the appropriate surgical device (stopcock on Trocar or Canula).
2. Connect large open end of tubing to Buffalo Filter® Smoke Evacuator filter inlet port. If fluid will be evacuated, please use fluid suction container (P/N SCC1001).
3. Position the drape clip.
4. When smoke plume needs to be evacuated, depress the suction on/off button on the Buffalo Filter® Smoke Evacuation System and then depress the valve as required to maintain a clear field.
5. This Laparoscopic Smoke Evacuator Tubing Kit should be used at approximately 10-50% suction control level with any Buffalo Filter Smoke Evacuation System.

CAUTION: Use the lowest possible setting for removal of smoke plume from the surgical site. Do not exceed 50% suction control level during surgical procedure.

CAUTION: Extended smoke removal by the stopcock on the trocar may cause reduction of the pneumoperitoneum.

Buffalo Filter®
Division of Medtek Devices, Inc.
595 Commerce Drive, Buffalo, NY 14228
www.buffalofilter.com
(716) 835-7000 OR (800) 343-2324 OR (716) 835-3414 fax
QF7-5-1DFU_BLAP100/100V.RevB

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Filiale de Medtek Devices, Inc.