

# BF EY04PF

## EY Prefilter

### INSTRUCTIONS

This prefilter product is to be used for prefiltration of smoke plume generated during laser procedures.

The installation/removal of the Buffalo Filter® EY Prefilter (P/N BFEY04PF) is simple to perform. The Buffalo Filter EY Prefilter is to be used with the ViroSafe® EY Filter (P/N VSEY1-4) and the PlumeSafe® Whisper™ TURBO Smoke Evacuator System (P/N PSWTURBO) during laser procedures.

**NOTE:** Failure to use the Buffalo Filter EY prefilter will dramatically shorten the 12-hour life of the ViroSafe® EY Filter.

**CAUTION:** Before installing or removing any filter, be sure that the system is turned off and the electrical plug has been disconnected.

**CAUTION:** If fluid is expected to be introduced into the ViroSafe® EY Filter, the Buffalo Filter EY Prefilter or the smoke evacuation system, fluid collection device (BF P/N SCC1001) must be installed with the vacuum hose assembly. Failure to install a fluid collection device could cause filter blockage and electrical damage.

### See drawing on reverse for additional information and instructions.

Please follow these instructions for installing/removing the prefilter:

1. Remove the Buffalo Filter EY Prefilter from the shipping box and remove any protective wrapping. Examine all filters for damage during shipping and storage. If damage is found, please contact the local shipper for damage assessment.
2. Install the ViroSafe® EY Filter according to the instructions that accompany that product.
3. Insert a Buffalo Filter EY Prefilter into the port on the ViroSafe® EY Filter by inserting the clear end of the prefilter into the ViroSafe® EY Filter to assure complete sealing.
4. Attach the Buffalo Filter EY Hose Kit (P/N VTEYKIT) according to the instructions that accompany that product.
5. Proceed with the proper use of the smoke evacuation system as outlined in the Operator's Manual.
6. After the Buffalo Filter EY Prefilter has been exhausted and requires changing, turn smoke evacuation system off and disconnect electrical cord from electrical socket.
7. Remove the Buffalo Filter EY Prefilter and dispose of in accordance with local codes.

For ordering information:

**Buffalo Filter®**

595 Commerce Drive  
Buffalo, New York 14228

(716) 835-7000

(800) 343-2324

(716) 835-3414 FAX



# BUFFALO FILTER<sup>®</sup> PLUMESAFE<sup>®</sup> *Whisper*<sup>™</sup> TURBO

## **SPECIAL PRODUCTS FOR FILTRATION OF SMOKE PLUME DURING LASER PROCEDURES**


### ViroSafe<sup>®</sup> EY Filter



Lasts for 12 Hours of Laser Time  
**Cannot be used/purchased  
 without prefilters**  
 (use without prefilters will  
 dramatically decrease life of filter)


Once filter life indicates 6 hour  
 timed replacement – reset filter life  
 for next 6 hours to complete 12  
 hour filter life

### EY Prefilter

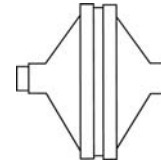

Lasts for 1-3 Procedures  
 These filters can be reordered  
 four per case  
**Reorder P/N: BFEY04PF**

### The EY Filter Kit Part Number VSEY1-4 (One EY Filter and 4 Prefilters)

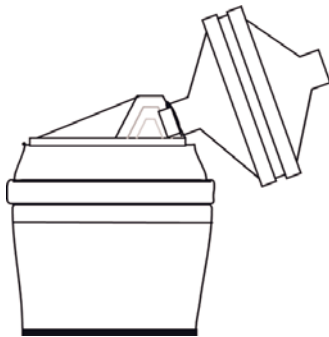

Insert clear end connector on prefilter into ViroSafe EY Filter

- Change prefilter after 1-3 procedures
- Change EY filter after 12 hours of operating time

### EY Hose Kit (Custom For EY Prefilters) Part Number: VTEYKIT



**Each kit contains:**

- (1) 1-3/8" x 6' (35mm x 183 cm) Hose
- (6) 1-3/8" x 2' (35mm x 61 cm) Hose with connectors, Integral Wand and Sponge Guard



Attach 6' (183 cm) hose to prefilter, attach 2' (61 cm) hose to  
 connector on end of 6' (183 cm) hose – replace 2' (61 cm)  
 hose after each procedure