

BFPF7/8-10

Prefilter

For the pre-filtration of smoke plume generated during surgical procedures

INSTRUCTIONS

BFPF7/8 Prefilter can be used with any ViroSafe[®] Filter with a 7/8" port.

Precautions:

1. Single use only. Remove and discard prefilter after each surgical procedure.
2. Before installing or removing any filter, disconnect system from electrical outlet.
3. Install a fluid collective device (BF P/N SCC1001) if excessive fluid is expected. Failure to install a fluid collection device could cause filter blockage and electrical damage.

Instructions:

1. Insert the BFPF7/8 Prefilter into the 7/8" port of the ViroSafe[®] Filter. The prefilter label should be facing toward user. Press in completely to assure complete sealing.
2. Attach a 7/8" ID vacuum hose to the 7/8" male connector of the prefilter. Installation is complete.
3. At the end of the surgical procedure, disconnect smoke evacuator from electrical outlet. Carefully remove BFPF7/8 prefilter from the ViroSafe Filter.
4. Dispose of BFPF7/8 prefilter in accordance with local codes. Used prefilters should be considered as potentially infectious waste.

For ordering information:

Buffalo Filter[®]

595 Commerce Drive
Buffalo, New York 14228

(716) 835-7000

(800) 343-2324

(716) 835-3414 FAX

