INSTRUCTIONS FOR USE

PLEASE CHECK PRODUCT LABEL AND FOLLOW APPROPRIATE INSTRUCTIONS BELOW

Sterile Unless Unit Package Damaged or Opened – Single Use – Disposable

Laparoscopic Smoke Evacuator Tubing Kit Part Number BLAP100

1. Connect patient end Luer Connector on the tubing to the appropriate surgical device (stopcock on trocar or canula).
2. Connect large open end of tubing to Buffalo Filter® Smoke Evacuator filter inlet port. If fluid will be evacuated, please use fluid suction container (P/N SCC1001).
3. Position the drape clip.
4. When smoke plume needs to be evacuated, depress the suction on/off button on the Buffalo Filter® Smoke Evacuation System and then turn the stopcock on the trocar to evacuate the smoke from the surgical site.
5. This Laparoscopic Smoke Evacuator Tubing Kit should be used at approximately 10-50% suction control level with any Buffalo Filter Smoke Evacuation System.

CAUTION: Use the lowest possible setting for removal of smoke plume from the surgical site. Do not exceed 50% suction control level during surgical procedure.

CAUTION: Extended smoke removal by the stopcock on the trocar may cause reduction of the pneumoperitonium.

Laparoscopic Smoke Evacuator Tubing Kit With Control Valve Part Number BLAP100V

1. Connect patient end Luer Connector on the tubing to the appropriate surgical device (stopcock on Trocar or Canula).
2. Connect large open end of tubing to Buffalo Filter® Smoke Evacuator filter inlet port. If fluid will be evacuated, please use fluid suction container (P/N SCC1001).
3. Position the drape clip.
4. When smoke plume needs to be evacuated, depress the suction on/off button on the Buffalo Filter® Smoke Evacuation System and then depress the valve as required to maintain a clear field.
5. This Laparoscopic Smoke Evacuator Tubing Kit should be used at approximately 10-50% suction control level with any Buffalo Filter Smoke Evacuation System.

CAUTION: Use the lowest possible setting for removal of smoke plume from the surgical site. Do not exceed 50% suction control level during surgical procedure.

CAUTION: Extended smoke removal by the stopcock on the trocar may cause reduction of the pneumoperitonium.

Buffalo Filter®
Division of Medtek Devices, Inc.
595 Commerce Drive, Buffalo, NY 14228
www.buffalofilter.com
(716) 835-7000 OR (800) 343-2324 OR (716) 835-3414 fax
QF7-5-1DFU_B LAP100/100V.RevB

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Filiale de Medtek Devices, Inc.