

BSPF030

Laser and Electrosurgery
Smoke Plume Prefilter

INSTRUCTIONS

The Buffalo Filter Laser and Electrosurgery Smoke Plume Prefilter is designed to be a replacement filter (Buffalo Filter P/N BSPF030) for the Surgimedics® Surgifresh™ Prefilter (Surgimedics P/N 901202-000). This filter is used with all Surgimedics Surgifresh Smoke Filtration Systems: Surgimedics Mini and Surgimedics Plume-inator. This filter can also be used in ValleyLabs AirForce Smoke Evacuation System.

1. Turn unit off and unplug the system from electrical power.
2. Remove the prefilter from the top of the unit.
3. Dispose of the filter according to the protocol of your facility.
4. Install a Buffalo Filter Replacement Prefilter, BSPF030, by placing the prefilter securely into the top of the ULPA/Carbon Laser Smoke Plume Filter, part number 901009-010. If you are using an older version of the smoke evacuation unit, the prefilter can also be directly installed into the top opening of the smoke evacuator.
5. Proceed with the installation of an ULPA/Carbon Replacement Filter if the filter change indicator shows that the ULPA filter needs to be changed. See the instructions for the changing of the ULPA/Carbon Filter for more details.

Buffalo Filter Replacement Prefilter (P/N BSPF030) is designed to be changed after every use.

**For ordering information:
Buffalo Filter®
595 Commerce Drive
Buffalo, New York 14228**

**(716) 835-7000
(800) 343-2324
(716) 835-3414 FAX**



QF7-5-1 DFU BSPF030.doc