

BTLD001 AND BTLD004

TeleVac® Filter

Berchtold®/Buffalo Filter®

Reorder Number: BTLD004

For the filtration of smoke plume generated during surgical procedures

INSTRUCTIONS

PRECAUTION:

- Before installing or removing any filter, be sure that the smoke evacuator system motor is turned off.
- Install a fluid collective device (BF P/N SCC1001 or SCC1002) if excessive fluid is expected. Failure to install a fluid collection device could cause filter blockage and electrical damage

DIRECTIONS:

1. Install the filter into the round opening on the front of the smoke evacuation module of the Berchtold Televac. Assure proper seating of the filter. The gasket on the bottom of the filter must contact to the back plate of the filter housing.
2. Attach adapters or hose onto the filter inlet port. Assure that good sealing is achieved; install other adapters, wands, cones, tubing, etc. as required.
3. At the end of the recommended use time, remove the filter from the smoke evacuation system and dispose of in accordance with local codes. Used filters should be considered as potentially infectious waste.

For ordering information:

Buffalo Filter®

595 Commerce Drive

Buffalo, New York 14228

www.buffalofilter.com

716.835.7000

800.343.2324

716.835-3414 FAX

