# PenAdapt™

REF/CAT: PA1025 PenAdapt with 3/8" x 1' Tube - Sterile 25/Case

REF/CAT: PA2010 PenAdapt with 3/8" x 4' Tubing Connected to a 7/8" x 6' Tube Sterile 10/Case

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800-343-2324

Tel.: +1-716-835-7000

**EN** **Indications For Use:** PenAdapt is an aspiration sheath that fits over an electrosurgical pencil body and leaves the tip or blade exposed. This device is considered an accessory to an electrosurgical unit (ESU). Use to remove smoke, particles, body fluid and casual fluids from the point of surgical activity during surgical procedures that use ESU for cutting and cauterizing. This device is used in conjunction with a suction (vacuum) source.

#### **Precautions:**

1. CAUTION: Federal Law (USA) restricts this device to sale by or on the order of a physician.
2. Sterile unless the unit package is opened or damaged.
3. Single use only. Do not reuse.
4. Discard according to hospital protocol for hazardous wastes.
5. If PenAdapt becomes clogged during use, dip the nose into sterile saline solution and begin suctioning. Pinch or gently knead the occluded area to loosen the blockage. If Penadapt continues to clog, discard and replace with a new unit.

**Contraindication:** This device is not to be used for microsurgery.

**Instructions for Use:** Using aseptic technique, open packaging and remove PenAdapt onto the sterile field.

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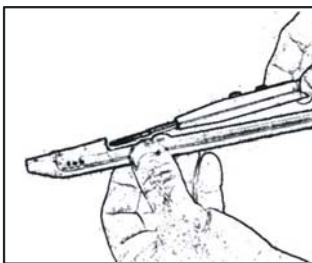
**Step 1** - Guide the ESU pencil with blade through the nose of PenAdapt by piercing the membrane and seat as deeply as possible.

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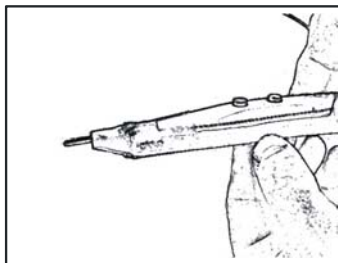
**Step 2** - Seat pen so that ESU blade protrudes approximately 3/4 of an inch through the inlet. (Penadapt nose may be trimmed to expose more blade length.)

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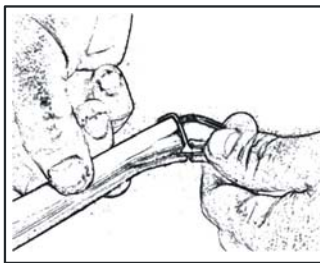
**Step 3** - Secure PenAdapt to the ESU pencil by gently stretching the sheath and positioning the uprights of the connector over the end of the pencil. The uprights should straddle the ESU pencil wire.

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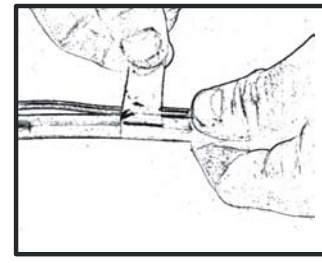
**Step 4** - Secure the wire of the ESU pencil to the medical tubing with Velcro strap(s) provided.

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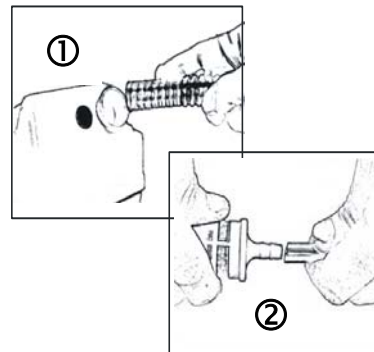
**Step 5** - Connect the end of the tubing to a vacuum source or smoke evacuator capable of at least 5 inches of mercury vacuum. This is required for efficient aspiration using PenAdapt.

PA2010 includes tubing for connection to filter and vacuum source (see ①). PA1025 product requires additional tubing for connection to filter and vacuum source (see ②).

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#### LIMITED WARRANTY, LIMITATION OF LIABILITY AND DISCLAIMER OF OTHER WARRANTIES:

Buffalo Filter warrants that reasonable care was used in the manufacture and production of this product. Because Buffalo Filter has no control over the conditions of use, patient selection, surgical procedure, post surgical stresses, or handling of the device after it leaves our possession, Buffalo Filter, does not warrant either a good effect or against an ill effect following its use. Buffalo Filter shall not be responsible for any incidental or consequential loss, damage or expenses directly or indirectly arising from the use of this product. Buffalo Filter's SOLE responsibility, in the event Buffalo Filter determines the product was defective when shipped by Buffalo Filter shall be replacement of the product. This warranty is in lieu of and excludes all other warranties not expressly set forth herein, whether express implied by operation of law or otherwise, including but not limited to, any implied warranties of merchantability or fitness for use.

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