

VTEYKIT

EY Hose Kit

INSTRUCTIONS

These products are to be used with the ViroSafe® EY Filter and the Buffalo Filter® EY Prefilter for filtration of smoke plume generated during Erbium:YAG Laser procedures.

The installation/removal of the Buffalo Filter® EY Hose Kit (P/N VTEYKIT) is simple to perform. The Buffalo Filter EY Hose Kit is to be used with the Buffalo Filter EY Prefilter (P/N BFEY04PF) and the ViroSafe® EY Filter (P/N VSEY1-4). The entire setup of filter, prefilter and hose kit is to be used with the PlumeSafe® Whisper™ TURBO Smoke Evacuator System (P/N PSWTURBO).

NOTE: Failure to use the Buffalo Filter EY prefilter will dramatically shorten the 12-hour life of the ViroSafe® EY Filter.

CAUTION: If fluid is expected to be introduced into the ViroSafe® EY Filter, the Buffalo Filter EY Prefilter or the smoke evacuation system, fluid collection device (BF P/N SCC1001) must be installed with the vacuum hose assembly. Failure to install a fluid collection device could cause filter blockage and electrical damage.

See drawing on reverse for additional information and instructions.

Please follow these instructions for installing/removing the Hose Kit:

1. Remove the Buffalo Filter EY Hose Kit from the shipping box and remove any protective wrapping. Examine each Hose Kit for damage during shipping and storage. If damage is found, please contact the local shipper for damage assessment.
2. Install the ViroSafe® EY Filter according to the instructions that accompany that product.
3. Install the Buffalo Filter EY Prefilter according to the instructions that accompany that product.
4. Install the single non-sterile 1 3/8" x 6' (35mm x 183cm) Vacuum Hose into the port on the prefilter.
5. Attach a single use non-sterile 1 3/8" x 2' (35mm x 61cm) Vacuum Hose (which includes the hose connector with integral wand and protective tip) to the 1-3/8" x 6' Vacuum Hose.
6. Proceed with the proper use of the smoke evacuation system as outline in the Operator's Manual.
7. After each procedure the 1 3/8" x 2' (35mm x 61cm) vacuum hose with connector will need to be changed. Turn smoke evacuation system off and disconnect electrical cord from electrical socket.
8. Remove the 1 3/8" x 2' (35mm x 61cm) vacuum hose and connector and dispose of in accordance with local codes.

For ordering information:
Buffalo Filter®
595 Commerce Drive
Buffalo, New York 14228

(716) 835-7000
(800) 343-2324
(716) 835-3414 FAX



BUFFALO FILTER[®] PLUMESAFE[®] *Whisper*[™] TURBO
SPECIAL PRODUCTS FOR FILTRATION OF SMOKE PLUME
DURING LASER PROCEDURES


ViroSafe[®] EY Filter



Lasts for 12 Hours of Laser Time
Cannot be used/purchased without prefilters
 (use without prefilters will dramatically decrease life of filter)

Once filter life indicates 6 hour timed replacement – reset filter life for next 6 hours to complete 12 hour filter life


EY Prefilter

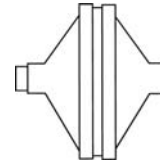

Lasts for 1-3 Procedures
 These filters can be reordered four per case
Reorder P/N: BFEY04PF

The EY Filter Kit
Part Number VSEY1-4

(One EY Filter and 4 Prefilters)

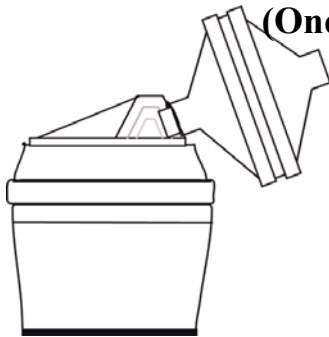

Insert clear end connector on prefilter into ViroSafe EY Filter

- Change prefilter after 1-3 procedures
- Change EY filter after 12 hours of operating time

EY Hose Kit (Custom For EY Prefilters)
Part Number: VTEYKIT



Each kit contains:

- (1) 1-3/8" x 6' (35mm x 183 cm) Hose
- (6) 1-3/8" x 2' (35mm x 61 cm) Hose with connectors, Integral Wand and Sponge Guard



Attach 6' (183 cm) hose to prefilter, attach 2' (61 cm) hose to connector on end of 6' (183 cm) hose – replace 2' (61 cm) hose after each procedure