

**ViroSafe® OH Filter – 24 Hours Filter Life**  
**ViroSafe® XLL Filter – 18 Hours Filter Life**  
**ViroSafe® VET Filter – 24 Hours Filter Life**

**INSTALLATION/REMOVAL INSTRUCTIONS**

**CAUTION: Before installing or removing any filter, be sure that the system main power is turned off.**

1. Remove the filter from shipping box and any protective wrapping. Examine filters for damage. If damaged, contact local shipper for damage assessment.
2. Depending on the model of Buffalo Filter PlumeSafe Smoke Evacuation System being used, install the ViroSafe filter into the square opening at the top, back or front of the smoke evacuation system. Be sure that the filter is seated completely into the opening and that the filter is resting on the bottom plate.
3. Insert adapter or hose into round filter opening. Be sure sealing is achieved. Add any other adapter, wand, tubing, etc. that will be needed to complete the installation.

**CAUTION: If fluid is expected to be introduced into the ViroSafe Filter and/or either smoke evacuation system, fluid collection devices (BF P/N SCC1001) must be installed with the vacuum hose assembly. Failure to install a fluid collection device could cause filter blockage and electrical damage.**

4. Connect smoke evacuator electrical cord into available grounded 120V/50/60Hz (220/240V/50/60Hz Int'l) electrical wall socket.
5. Proceed with the proper use of the smoke evacuation system and ViroSafe filter as outlined in the Operator/Service Manual.

**FILTER LIFE RESET – TIMED FILTER LIFE**

6. If your PlumeSafe Smoke Evacuator is equipped with RFID Intelligence (label located on top of the filter and on the smoke evacuation system) and the electronic keypad **DOES NOT HAVE A FILTER RESET BUTTON** – the smoke evacuation system will automatically adjust the timer for the correct filter life, proceed to step #9.
7. Some PlumeSafe Smoke Evacuators have a built-in filter life timer of **six (6) hours and the electronic keypad HAS A FILTER LIFE RESET button**. These systems are **NOT** automatic. After initial installation of the filter, press the FILTER LIFE RESET button on the membrane key panel and hold for 5 seconds until all indicator lights are on. After all indicator lights remain constant release button. This will reset the filter life indicator.
8. Use the check boxes located on the top of the filter to assist in tracking your filter life after every six hours (the red filter life reset LED will “blink” after each six hour period). Simply reset your filter life by pressing the reset button and holding for 5 seconds to track your next six hour filter life. Once you have checked all boxes, your filter life has been exhausted and requires changing. Turn smoke evacuation main power off.
9. Remove the ViroSafe filter from the smoke evacuation system when no filter life is remaining. Dispose of in accordance with local codes.

We trust that you have been satisfied with this product. Should you have any questions, please call us at:

**BUFFALO FILTER®**  
Buffalo, New York 14228  
(716) 835-7000  
(800) 343-2324  
(716) 835-3414 FAX  
[www.buffalofilter.com](http://www.buffalofilter.com)

